



Care Instruction for Patients Wearing Snap-On Smile™ Appliances

1. **ALWAYS** remove the appliance by gently rocking it in a side to side motion *using both hands*.
2. **DO NOT** sleep with the appliance unless it was designed for use as a Bruxguard, Nightguard, or Ortho Retainer.
3. **RINSE** the appliance after meals.
4. **BRUSH** both inner and outer aspects of the appliance twice a day to keep your Snap-On Smile™ clean.
5. **ALWAYS** store your appliance in the container designed for it to protect it from becoming lost or damaged.
6. **NEVER** squeeze or flex the appliance.

For any additional information or requests please contact your dental professional or call (877) 7-SnapOn or (877) 776-2766